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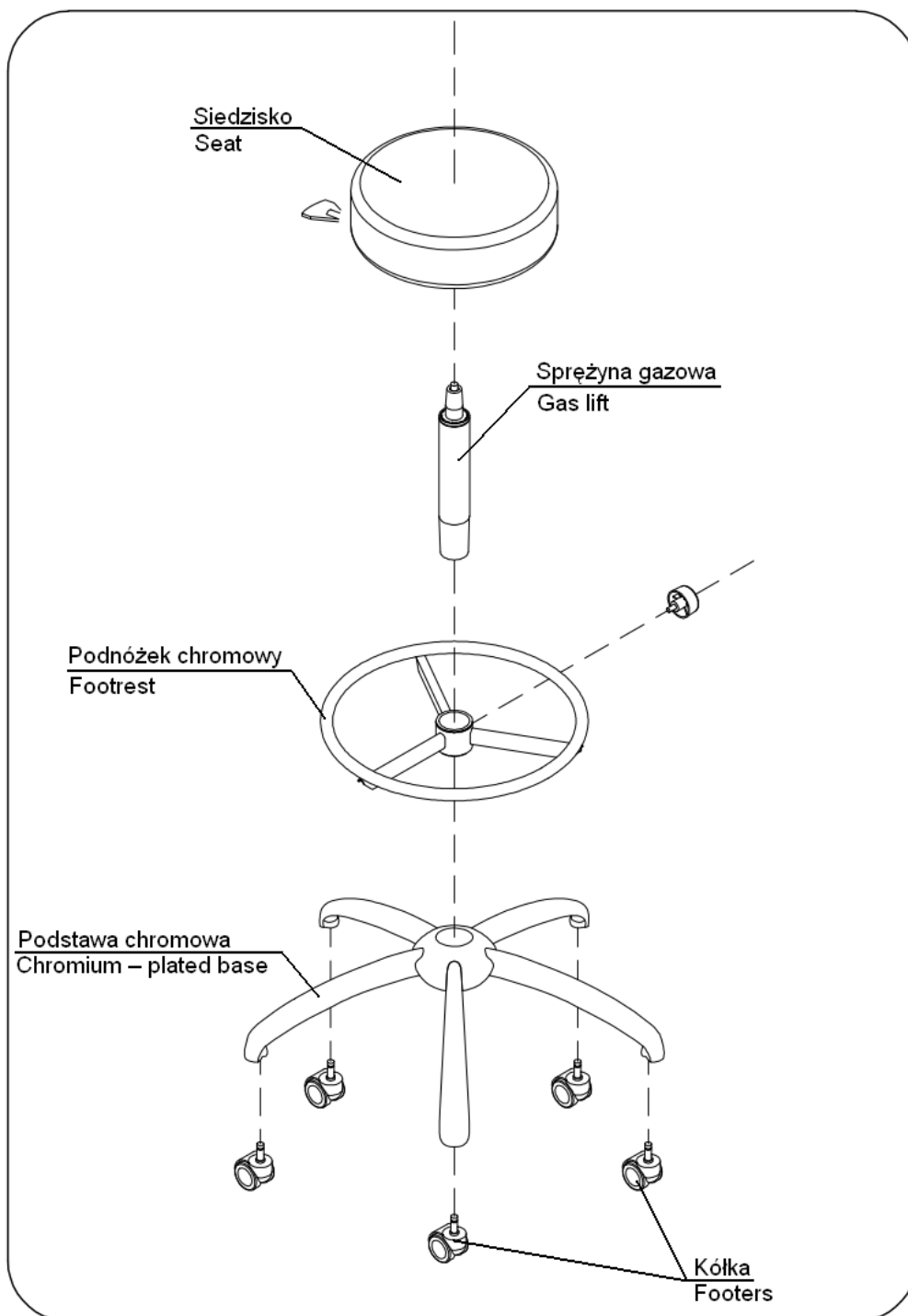
## I. Technical details

Omega - chromium - plated base on wheels with a footrest  
base - fi 640 mm  
height adjusted: 600 - 790 mm

## II. Design

Omega - stool with a chromium – plated base on rubber wheels with a diameter of 640 mm.  
Height adjusted with a range of 600 – 790 mm with a gas lift. It has a chromium – plated adjustable footrest

## III. Operation

Height adjustment is easy to operate thanks to the gas lift. Additionally, the height of the footrest can be adjusted.

## IV. Maintenance

Maintenance involves washing or cleaning the upholstery and table structure with a wet cloth, using commercially available cleaning agents for dermatoid materials and varnished wood. Do not use rough chemicals like dissolvents, petrol or acetone, as they may permanently damage or discolor the upholstery and wood varnish! The stool may be disinfected with generally used agents based on alcohol, chloramine, aldesan. These agents should be used with appropriate permits.

## V. Recommendations

1. Use the stool according to these instructions.
2. Read and observe the terms of the guarantee.
3. Service your stool after two years of use.

## VI. Components