Instruction manual & warranty for stationary massage tables

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Ultima

Please read the manual and the included safety instructions before using the table.

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strona 9

I. INTRODUCTION

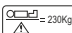
HABYS Ultima stationary tables enable physiotherapists and massage therapists to correctly, safely and effectively carry out rehabilitation, physiotherapy, physical therapy, massage treatments to alleviate the course of disease and the effects of injury/disability. The design of the HABYS Ultima rehabilitation tables allows the physiotherapist easy and full access to the patient. Based on a hydraulic cylinder, the lifting mechanism allows the full functionality of the tables to be used without the need for an electrical connection. Thanks to their design, the tables are also suitable for use by massage therapists performing sports, beauty and relaxation massages.

II. INSTALLATION

1. Unpacking.

- Unpack the table.
- Check for damage during transport. If it were damaged, do not plug it in and notify the dealer immediately.
- Ensure that all accessories are complete. If, on delivery, discrepancies with the order are found, notify the seller immediately.
- Tables with electric controls should be stored and later used under normal dry conditions, in enclosed spaces where the temperature is controlled between + 10 and 40°C and the relative humidity is between 30 and 75°C.
- If the hydraulic control table will be exposed to temperatures below 0°C during storage, the recommended temperature should be set to at least 5 hours before use in the room.

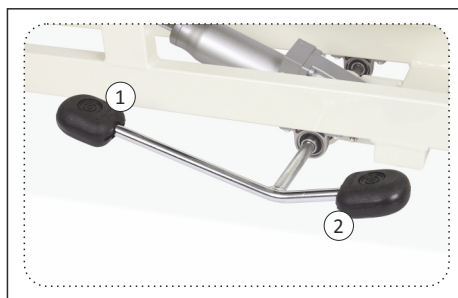
III. LEGEND TO THE PICTOGRAMS

	Attention		During storage, protect against moisture and wetness		Name and address of the manufacturer		Authorised distributor
	Maximum load		CE mark - compliance of the product with the requirements of the Medical Devices Regulation 2017/745/EU		Read the instructions for use		Production date
	Medical device		Importer				

IV. OPERATION

1. Height adjustment.

Raising and lowering of the tables is done by a foot lever:

The table is equipped with a foot lever for height adjustment.

Press lever 1 to raise the table top.
Press lever 2 to lower the table top.

2. Adjustable headrest and lower limb sections.

The adjustment of the headrest is carried out by a gas spring:

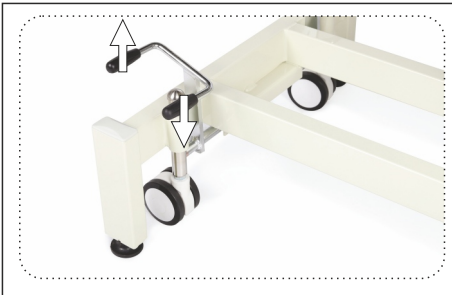
After releasing the lever, move the headrest / lower limb section to the desired position.

It is forbidden to sit on the headrest.

3. Running gear.

The table is equipped with a lever to control the wheels:

For mobility: place your foot on the positioning lever and push downwards.

To lock: lift the positioning lever with your foot up.

Do not use the foot lever system during treatment or to transport the patient. The option is only used to adjust the table layout.

V. RECOMMENDATIONS AND PRECAUTIONS

- Use the table in accordance with the operating instructions.
- Damage caused by non-compliance with the above remarks is not grounds for complaint.
- The equipment is not intended to be used by persons of reduced physical or mental capacity or lacking in experience and knowledge, unless they use it under supervision or after receiving instructions from a person responsible for their safety.
- The table should only be used by adults and in accordance with the instructions.
- The table is designed for indoor use in dry and confined spaces and for treatments for only one person at a time.
- Moving the table with the patient is not allowed.
- It is forbidden to sit down violently on the table (jumping in).

- Sitting, kneeling and leaning on the headrest, shelves and armrests is prohibited.
- If the patient is placed on the table without clothing, the rehabilitation therapist should use disposable pads or disinfect the table.
- Avoid prolonged contact of the table upholstery with oils. Oilproof coverings or waterproof sheets should be used for this purpose.
- The table should be used and stored away from direct sources of fire and heat.
- The table should not be exposed to prolonged sunlight. This can cause discolouration of the table upholstery.
- After bringing the table from the cold room, connect it to the mains after waiting 15 minutes.
- It is advisable to make a full table height adjustment from minimum to maximum to ensure even distribution of the lubricant inside the motor. This ensures proper table maintenance and extends the life of the motor. Repeat 2 or 4 times at least once a week.
- Do not remove the plug from the socket by pulling on the cable.
- Do not dismantle the device and attempt to access the interior.
- There are no contraindications to the use of the product if and only if the above recommendations are met.
- Any serious incident related to the device should be reported to the manufacturer and the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

VI. MAINTENANCE AND CLEANING

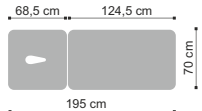
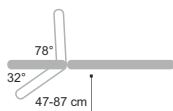
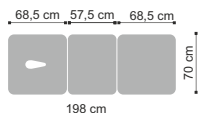
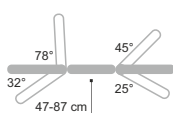
Steel and painted components: Non-abrasive products are recommended for cleaning such components.

Disinfection: Disinfect as necessary and occasionally. **We recommend that you read the additional instructions for use of Habys upholstery.**

In accordance with the provisions of the Waste Electrical and Electronic Equipment Act, it is prohibited to dispose of waste equipment labelled with the crossed-out wheeled bin symbol with other household waste. Return used equipment to the appropriate collection point. These statutory obligations were introduced to limit the amount of waste produced from waste electrical and electronic equipment and to ensure an adequate level of collection, recovery and recycling of waste equipment. The correct implementation of these obligations is of particular importance if the waste equipment contains hazardous components that have a particularly negative impact on the environment and human health.

VII. TECHNICAL DATA

	Dimensions	Possible positions	Weight	Maximum load
Ultima H2			62 kg	230 kg
Ultima H3			65 kg	230 kg

VIII. WARRANTY CONDITIONS

1. For the purposes of the guarantee:
 - a) "Manufacturer" refers to HABYS Spółka z ograniczoną odpowiedzialnością (Limited Liability Company) with its registered seat in Jasło, ul. Produkcyjna 16, 38-200 Jasło, entered into the Register of Entrepreneurs kept by the District Court in Rzeszów, XII Commercial Division of the National Court Register, with the assigned KRS number: 0000513317, with the share capital in the amount of PLN 10.750.000 (ten million seven hundred fifty thousand zlotys), the capital has been fully paid up, NIP: 6852208438, REGON: 180186290.
 - b) "Buyer" refers to the final Buyer of the equipment, manufactured by the Manufacturer, including both the consumer and the entrepreneur.
 - c) "Distributor" refers to an entity that, under an agreement with the Manufacturer, sells the Equipment in a designated area.
 - d) "Authorized Service Centre" refers only to a service held by the Manufacturer or a service indicated by the Manufacturer and held by its Distributor,
 - e) "Equipment" refers to a movable item, manufactured by the Manufacturer, being the subject of the sales and to which the said warranty applies.
2. **The manufacturer ensures good quality and efficient operation of the equipment, for which the warranty card has been issued, for the period of: 2 years for constructional elements, 2 years for upholstered elements and 2 years for electronic and hydraulic accessories.**
3. The guarantee repair does not include the activities provided for in the manual, which the Buyer is obliged to perform on his own (installation of the Equipment, maintenance of the Equipment, etc.).
4. Liability under the guarantee covers only defects resulting from causes inherent in the Equipment. In particular, the guarantee does not cover any damage resulting from improper or inconsistent use, storage, maintenance or transport of the Equipment, as well as colour differences between the parts exposed and not exposed to solar radiation, the influence of chemical or abrasive agents, atmospheric pollution or accidental punctures and abrasions, the presence of stains or rings from greasy or dyeing substances (fats, creams, lotions or oils), permanent staining with dyes such as inks, permanent markers or other non-permanent dyes used in the production of clothing (e.g. dyes used to make blue jeans).
5. The guarantee does not cover any mechanical damage to the Equipment.
6. The buyer loses the guarantee rights in the event of:
 - a) the Manufacturer states that the Equipment was repaired by entity other than the Authorized Service,
 - b) making any design modifications to the Equipment without the prior written consent of the Manufacturer,
 - c) using the Equipment contrary to its intended use,
 - d) using the Equipment after revealing any defect in it.
7. Regardless of the loss of the guarantee rights, described above, in point 6, the Manufacturer shall not be liable for any damage to property or caused to third parties, which may be caused by the use of the Equipment covered by the guarantee.
8. The guarantee does not exclude or limit the rights resulting from the warranty for defects. Exercising the rights under the guarantee does not affect the seller's liability under the warranty for defects. If the Buyer exercises the rights under the guarantee, the time limit for the exercise of the rights under the warranty for defects is suspended on the date the Manufacturer is notified of the defect. The period runs further from the date of the Manufacturer's refusal to perform the obligations arising from the guarantee or the ineffective expiry of the time for their performance.
9. The scope of the guarantee coverage is limited to the territory of the country in which the Equipment was sold.
10. The Manufacturer shall perform its obligations under the guarantee (removal of defects or damage to the structure of the Equipment arising from reasons inherent in the Equipment, and disclosed during the guarantee period) free of charge within 14 days from the date of delivery of the Equipment to the Manufacturer or the Authorized Service Centre. If the repair of the Equipment cannot be performed within the above-mentioned period, the Manufacturer shall inform the Buyer of the above, and also indicate the reasons for the delay and the expected date of fulfilment of the recognized guarantee request.
11. The use of the guarantee rights requires the total (cumulative) fulfilment of four conditions:
 - a) presenting a VAT invoice, receipt or other proof of purchase of the equipment to the Manufacturer, photographic documentation of the defective Equipment, together with a brief description of the reasons for submitting a guarantee claim,
 - b) reporting the defect within 14 (fourteen) days of its detection,
 - c) delivering the Equipment in original packaging at the expense of the Manufacturer, to the address indicated above in the guarantee card or to the address of the Authorized Service Centre,
 - d) designation of the place where the Manufacturer delivers the equipment after repairing it (limited to the country in which the Equipment was purchased).
12. The guarantee period is extended for the duration of the repair, starting from the date of delivery of the Equipment to the Manufacturer or to the Authorized Service Centre, to the date on which the Equipment was returned to the Buyer.
13. The responsibility of the Manufacturer or the Authorized Service Centre is to define the scope and the method of repairing the Equipment.
14. The guarantee is limited to the repair of the Equipment. The Manufacturer reserves the right to replace the Equipment with a non-defective one, if the repair exceeds the price of the Equipment indicated in the proof of purchase. Under the guarantee, the Buyer shall not be entitled to demand that the Equipment be replaced with a new one. The guarantee, in particular, does not cover liability for losses or damages incurred by the Buyer in connection with the fact that one is not able to use the Equipment.
15. The Manufacturer shall not be liable for any damage to the Equipment caused during the performance of the transport service by the delivery company until it is collected by the Manufacturer from the delivery company.
16. If the shipment of the Equipment was ordered by the Manufacturer, the Buyer is obliged to check the technical condition of the Equipment upon its receipt. If the Buyer finds that the Equipment has been damaged during the transport, the Buyer is obliged to:
 - a) immediately, not later than within 3 days from the date of receipt of the shipment, notify the Manufacturer of any damage to the Equipment that occurred during transport, in order to enable the Manufacturer to submit a complaint to the delivery company and pursue related claims,
 - b) provide the Producer with a complaint report drawn up between the Buyer and the delivery company.
17. The guarantee regulations contained in this document are the sole and exclusive guarantee rules for HABYS limited liability products.

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